

Intentional leak rates for masks and exhalation ports

Philips Respironics sleep therapy devices report and/or display **total leak** rates.

Total leak = intentional leak plus unintentional leak

This chart provides the intentional leak rates of our mask exhalation ports at various pressure settings.

Unintentional leak = total leak minus intentional leak

(from mouth, mask, etc.)

Mask/exhalation port	Pressure (cm H ₂ O)								
	2.5	5	10	15	20	25	30	35	40
TrueBlue	13	18	26	32	38	43	48	52	55
GoLife	13	18	25	31	36	41	45	48	52
Whisper Swivel 2	11	17	26	33	40	45	50	55	59
ComfortGel Blue EasyLife	12	18	26	32	37	42	46	50	54
ComfortFusion ComfortSelect ComfortClassic Profile Lite	13	18	26	31	37	41	45	49	52
ComfortGel Full ComfortFull 2	12	19	28	34	40	44	48	52	56
FullLife	13	19	26	32	37	42	46	50	53
FitLife	8	13	20	26	31	35	40	43	47
OptiLife, all cushions	11	16	23	29	34	38	42	45	49
Small Child Profile Lite	20	31	45	57	68	74	82	89	96
ComfortLite 2, Simple Cushion	13	19	27	33	38	43	47	51	55
ComfortLite 2, pillows cushion and Direct Seal cushion	14	20	29	35	41	46	51	55	59

* The actual leak rate may vary due to slight differences in the manufacturing process.

Respironics, ComfortClassic, ComfortFull, ComfortFusion, ComfortGel, ComfortLite, ComfortSelect, Direct Seal, EasyLife, FitLife, FullLife, GoLife, OptiLife, Profile Lite, Simple Cushion, TrueBlue and Whisper Swivel are trademarks of Koninklijke Philips Electronics N.V. and its affiliates. All rights reserved.

©2011 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.

www.philips.com/respironics +1 724 387 4000 +1 800 345 6443